DÉMONSTRATION

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Canadian cPRA Calculator
Canadian Transplant Registry

The Canadian cPRA calculator is a component of the Canadian Transplant Registry (CTR), a web-based application used by the transplant community, to estimate the percentage of Canadian deceased organ donors with whom a transplant candidate may be incompatible.

This calculator uses the same formula and data as the CTR. It produces a value by comparing the unacceptable antigens entered below. If you are a transplant candidate, and have questions regarding the value generated by this calculator, please contact your transplant physician or coordinator.

Attention: The actual cPRA value assigned to a transplant candidate is calculated by the CTR based on the unacceptable antigens that are entered by the laboratory at the transplant candidate's transplant centre. The value produced by the cPRA calculator on this website is for your informational use only.

Region(s)

Alberta Atlantic Canada British Columbia Manitoba
 Ontario Quebec Saskatchewan

Blood Group(s)

A AB B D

A

1 2 3 11 23 24 25 26 29 30
 21 22 27 28 32 33 34

DQA

1 2 3 4 5 6

DQB

2 4 5 6 7 8 9

Calculate an ABO-Adjusted cPRA

Patient's Blood Group:

Decimals:

Canadian cPRA Calculator Version 5.0.12.13
Copyright Canadian Blood Services - All Rights Reserved

PRA-CALCULATOR.CA

Canadian cPRA Calculator Assistant

Enter Data Here

Class I or II values
Multiple text fields are only provided for convenience; you may enter values in either field.

Enter values

Enter values

Select patient's blood group:

Decimal places for calculation results:

Calculation Results

Please enter values, then click Process to calculate the results.

About the cPRAAssistant

The cPRAAssistant is a tool to assist with the use of the Canadian cPRA Calculator, which was built by Canadian Blood Services for the Canadian Transplant Registry. It is available, at this time, as a tool for the use of Canadian HLA laboratories, for their internal purposes.

Copy and paste your results into one or both of the text fields, then click on Process. Once the results are processed, you will be presented with a hypertext, which will take you to the selected cPRA Calculator.

You may paste your results in many formats, for example, the following are all acceptable:
A1 & 211034, B9: 21 D7 58
(the example above will result in: A1 A2 A11 A34 B7 B35 d06, duplicates are ignored)

General Information

This application makes use of the Canadian cPRA Calculator located at: www.cbc.ca/ctr/ctr/cra

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